## PERFORMANCE EVALUATION: OXYGEN CONCENTRATORS

Date: Lab Clinical				
Lab: Pass Fail	Clinical: Pass			
Student name	Instructor name			
No. of times observed in clinical				
No. of times practiced in clinical				
PASSING CRITERIA: Obtain 90% or better on the pr the evaluation is terminated. I performance receives a failing	Procedure must be perfo	by * must re rmed within	eceive at lea designated	st 1 point, or I time, or the
SCORING: 2 points — Task performed sati 1 point — Task performed sati 0 points — Task performed ind NA — Task not applicable to the	sfactorily with self-initiat correctly or with prompti	ed correctior	n.	
TASKS:		PEER	LAB	CLINICAL
1. Places the concentrator correctly at the po	oint of use		_	
* a. Away from walls or draperies				
* b. Away from heaters, heat registers, or	radiators			
<ol> <li>Connects the power cord, ensuring that the minimal electrical load</li> </ol>	he circuit has			
* 3. Checks the gross particle filter and cleans	as needed			
* 4. Connects a humidifier as required				
* 5. Connects the oxygen delivery device				
* 6. Adds up to 50 feet of extension tubing as	needed			
* 7. Turns on the power switch				
* 8. Sets the liter flow rate				
* 9. Documents procedure in the patient reco	rd		Ш	
SCORE: Peer points of possible				
Lab points of possible				
Clinical points of possible	20;%			
TIME: out of possible 30 minutes				
STUDENT SIGNATURES	INSTRUCTOR SIG	NATURES		
PEER:	LAB:			
STUDENT:	CLINICAL:			